

GoDirect™ Overdenture Options


Overdenture components are available to expand the treatment options of GoDirect™ implants. The screw-receiving top (4500-61) and ball top (4500-71) convert the existing overdenture attachment platform into one for use with either a screw-retained or ball attachment prosthesis, respectively.

The screw-receiving top is packaged with a white transfer (1000-68) and comfort cap (4000-13). With a profile similar to our other screw-receiving abutments, this top may be used with any of our existing screw-receiving abutment components.


The ball top is packaged with a red transfer (1000-84) and comfort cap (1000-87). This ball top has a similar profile to our other ball abutments and may be used with any of our existing ball abutment components.

To attach either top, the small, internal GoDirect™ screw must be removed using a 1.25mmD hex tool. This allows the abutment tops to engage with the internal threads of the implant.

Both tops are available at a US list price of \$85.

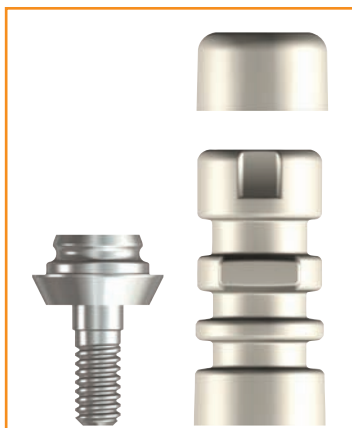

GoDirect™ Screw-Receiving
Top (4500-61)

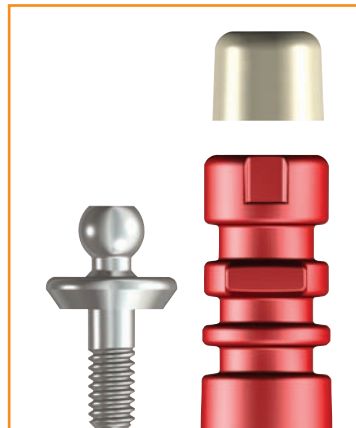

GoDirect™ Ball Top
(4500-71)

See these components in action!

[Watch Tutorial](#)



GoDirect™ implant with internal screw removed and
Screw-Receiving Top or Ball Top then attached