
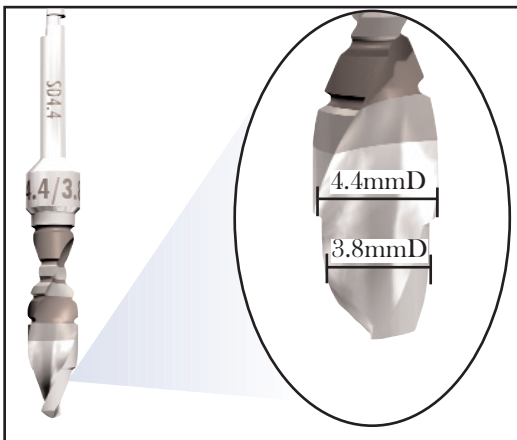


## Surgical Drills with Variable Band Depth (VBD) markings

### Using the surgical drills:

Reusable drills are designed to be used with external irrigation with a surgical unit that can supply a range of drilling speeds and with sufficient torque. A recommended range for drilling is between 600-1250 rpm, although clinicians may vary from this range in their protocol.

**Note:** The markings on the drills are in excess of the length of the implant to be placed by up to 1.0mm (13mmL is actually 14mmL). This added length is to accommodate for the design of the drill point.

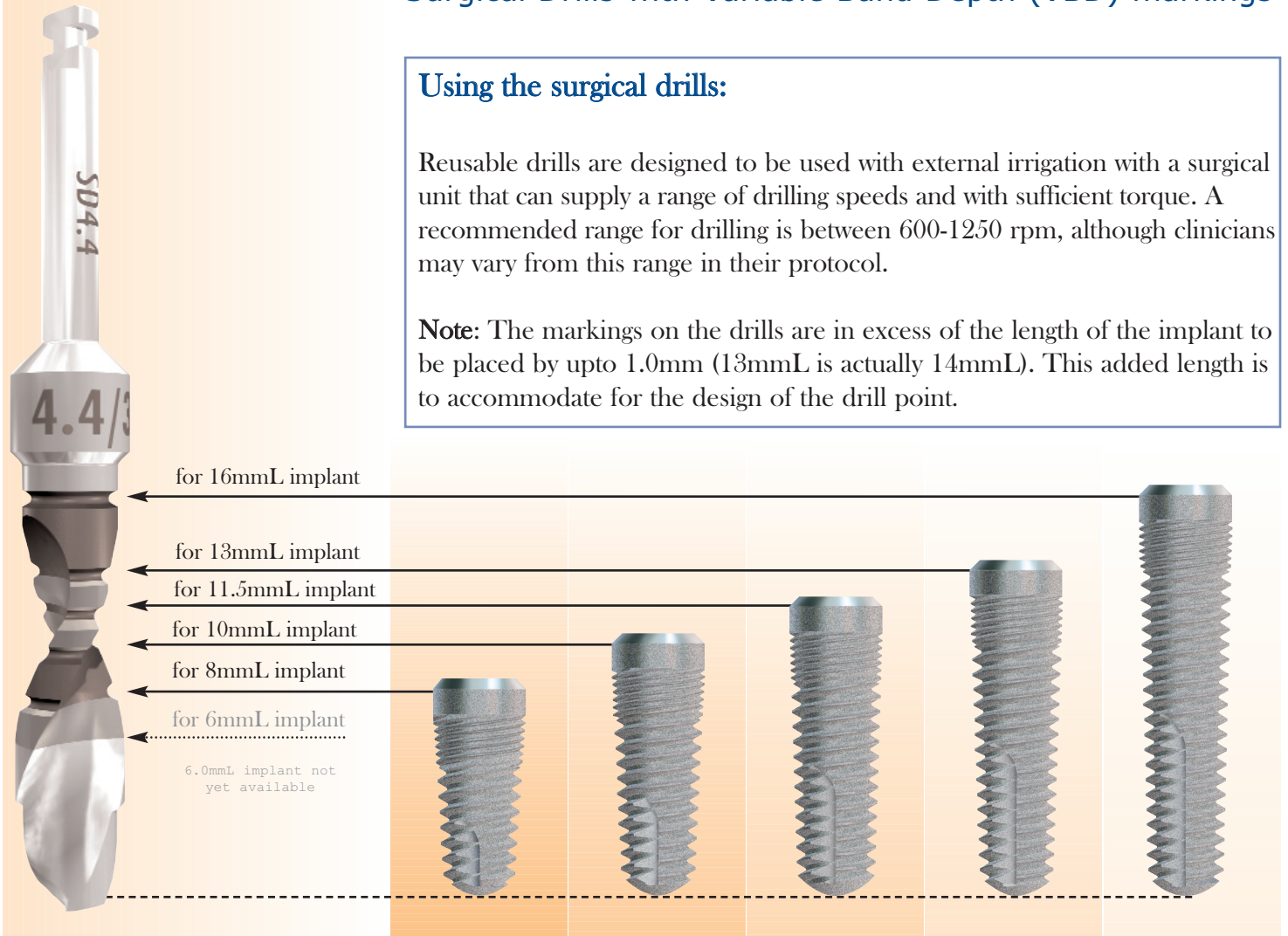

### Using step drills for tapered implants:

Use step drills for final sizing of the osteotomy when placing tapered implants in dense bone. These drills are designed to accommodate the varying lengths of tapered implants without having to have length specific tapered drills. The drill has two diameters of straight walled design incorporated into one drill. This allows the implants to obtain maximum engagement into bone no matter the length of implant being used. The length of the stepped area is approximately 4mm from the point of the drill to the start of the wider portion.

27030 Malibu Hills Road, Calabasas Hills, CA 91301

Phone: 818.444.3333 opt2: 888-NIZNICK (649-6425) Fax: 818.444.3400

[www.implantdirect.com](http://www.implantdirect.com)