



The RePlant, RePlus and ReActive Implants are mounted on color-coded fixture-mounts, designed for use as implant level transfers or shortened to function as an abutments. A cover screw is included in each vial. The RePlus fixture-mount is provided with a snap-on comfort cap.



Prepare surgical site with double cutting step drill



All of Implant Direct's implants come in a sterile vial



Each label is color-coded with two tear-off labels



Each cap is color-coded by platform diameter of implant



Drop implant assembly out of vial onto a sterile surface



Remove implant assembly with a fixture-mount drill or ratchet




Insert implant into osteotomy



Index fixture-mount flat to labial



Remove fixture-mount

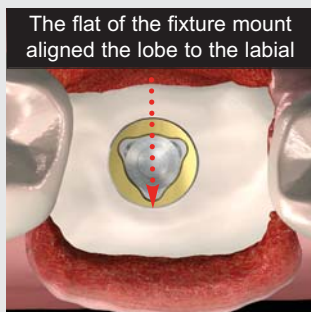

The flat of the fixture mount aligned the lobe to the labial



Remove the holder containing the cover screw & comfort cap



Snap-on comfort cap is located in the top portion of holder



Attach fixture-mount to corresponding color-coded implant analog




Remove colored section from fixture-mount through groove to create a final straight abutment



To attach abutment, use 30Ncm in hard bone. In soft bone, use only finger pressure



Check for occlusal clearance prior to attaching comfort cap

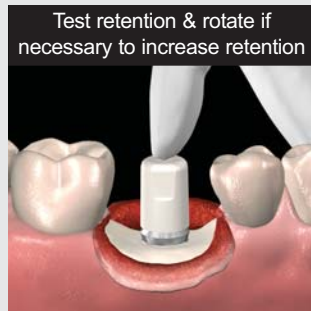

Test retention & rotate if necessary to increase retention



Suture soft tissue around comfort cap