

Subject:

Product Release: Nylon Liner Transfers

Date: 4-09-08


To: Implant Direct Customers

The Nylon Liner Transfer [1000-83] is now available!!!!

The Nylon Liner Transfers are used in the alignment of the metal Ball Abutment Housing [1000-81] on the Ball Abutment in the patient's mouth or on the Ball Abutment Analog [1000-85] in the working cast in the laboratory.

Press-fit the red Nylon Liner Transfer onto the Ball Abutment head. The red component has a retentive feature to engage the Ball head but is passive with regard to the metal housing. Place the Ball Abutment Housing onto the Nylon Liner Transfer. Rotate the assembled housings and transfers up to 28° to create relative parallelism for a common path of draw. Block out the undercuts beneath the housing assemblies with an appropriate material. Process the Ball Abutment Housing into the denture or baseplate (lab procedure).

The USA list price for the Nylon Liner Transfers is \$15.00 and are packaged two per vial.

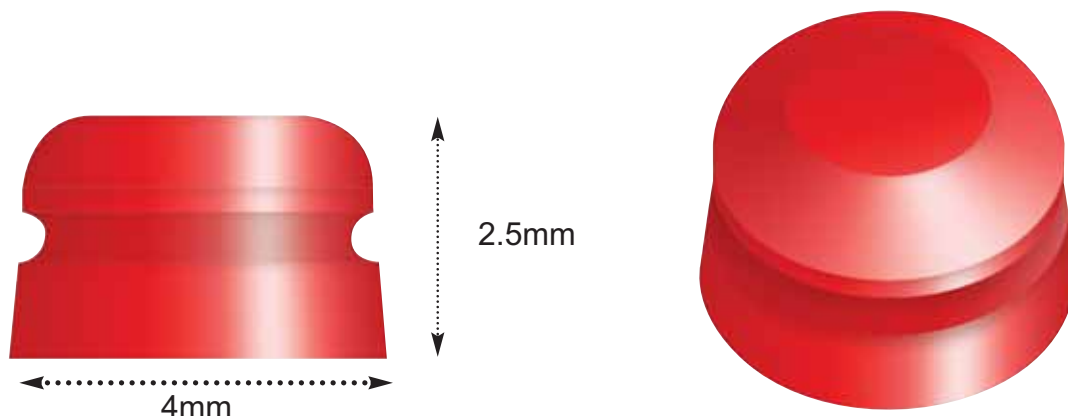

27030 Malibu Hills Road, Calabasas Hills, CA 91301

Phone: 818.444.333 opt2: 888-NIZNICK (649-6425) Fax: 818.444.3400

w w w . i m p l a n t d i r e c t . c o m

Ball Abutment Flow Chart

